





Restoration Treatment*


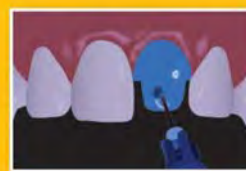






<p>Glass Ceramics/porcelain (silica-based) or Lithium Disilicate/Silicate</p>  <p>1. Apply PORCELAIN PRIMER to restoration</p>  <p>2. After try-in, decontaminate the internal surface</p>	OR	<p>Metal, Zirconia, Alumina, Composite</p>  <p>1. Apply Z-PRIME Plus to restoration</p>  <p>2. After try-in, decontaminate the internal surface</p>
---	----	---

NO LIMITS. NO BOUNDARIES.

DUO-LINK UNIVERSAL™

Tooth Preparation and Cementation*

Choose **SELF-ETCHING** or **TOTAL-ETCHING** technique below.

 <p>1. SELF-ETCHING: Wash thoroughly with water spray. Remove excess water (do not desiccate)</p>	OR	 <p>TOTAL-ETCHING: Prepare cavity. Wash thoroughly with water spray, apply phosphoric acid etchant and rinse thoroughly. Preparation should be visibly moist</p>	 <p>2. Apply ALL-BOND UNIVERSAL</p>	 <p>3. Air-dry</p>
 <p>4. Light cure for 10 seconds</p>	 <p>5. Apply DUO-LINK UNIVERSAL to restoration</p>	 <p>6. Seat the restoration</p>	 <p>7. Optional: Light cure</p>	

* Refer to complete instructions for detailed techniques.

ORDERING INFORMATION:

KIT CONTENTS

DUO-LINK UNIVERSAL SYSTEM KIT A-19620K

1 Dual-Syringe DUO-LINK UNIVERSAL Shade: UNIVERSAL (Base/Catalyst, 8g mixed), 1 Bottle ea. ALL-BOND UNIVERSAL (6ml), 1 Bottle Z-PRIME Plus (2ml), 1 Bottle Porcelain Primer (3ml), Accessories, Instructions/MSDS,

1 DUAL-SYRINGE PACKAGE

1 Dual-Syringe (Base/Catalyst, 8g mixed), 15 Mixing Tips and Intraoral Tips, Instructions/MSDS

UNIVERSAL A-19030P

MILKY WHITE A-197MWP

TO ORDER CALL:

Head Office: 1.800.667.8811

Français: 1.800.211.1200

www.biscocanada.com

Rx Only



Head Office: 1.800.667.8811

Français: 1.800.211.1200

www.biscocanada.com



OUT OF THIS WORLD STRENGTH!

DUO-LINK UNIVERSAL adhesive resin cement is specially formulated for cementation of ALL* indirect restorations. It is available in two esthetic shades, Universal and Milky White. **DUO-LINK UNIVERSAL** is intended for use with adhesives designed for compatibility with all dental materials, including all BISCO adhesives.

Universal - For all cementation procedures*

Crowns, bridges (fixed prostheses), inlays, onlays, and posts/dowels, fabricated from metal, composite, porcelain, ceramic, zirconia, alumina, CAD/CAM restorations, etc.

Extremely Easy Clean-Up

Formulated to allow for quick and easy removal of excess cement

Highly Radiopaque

Easily identified on radiographs for quick and effective diagnosis

High Degree of Conversion

High degree of conversion in both light- and self-cured modes ensures a strong, long lasting restoration

Easy to Use

Auto-mix, dual-syringe provides a consistent mix for immediate delivery


CAD/CAM restorations

Ideal for all chairside and lab-fabricated restorations

New shades

Universal shade has some translucency at low film thickness and Milky White shade is more opaque and can be used to mask darker tooth preps

Shear Bond Strength to Dentin (MPa)
Shear bond strength generated using Ultradent method on dentin

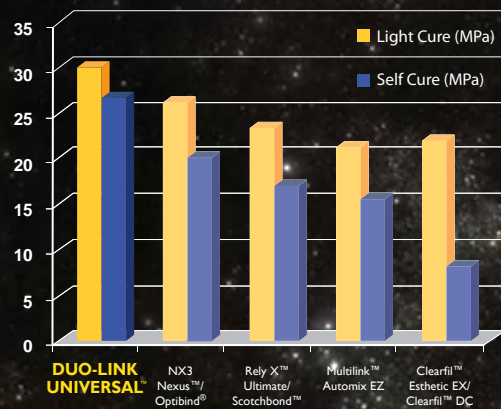

DUO-LINK UNIVERSAL outperforms other cements in shear bond strength.

Data on file, BISCO, Inc. Trademarks are property of their respective owners.

Courtesy of Adamo Notarantonio, DDS, AAACD, FICOI



DUO-LINK UNIVERSAL cement dispensed into crown



Seated crown



Final restoration



Easily identifiable on a radiograph

DUO-LINK UNIVERSAL SYSTEM KIT...

UNIVERSALLY SIMPLE, UNIVERSALLY STRONG



DUO-LINK UNIVERSAL KIT

is an adhesive cementation system designed to effectively address the cementation of all your indirect restorations for optimal performance and esthetics. The combination of award-winning, innovative products provides high retention and durability of indirect substrates in a simple, universal, and easy-to-use kit.

DUO-LINK UNIVERSAL™

DUO-LINK UNIVERSAL™ adhesive resin cement is specially formulated for cementation of ALL indirect restorations and provides all of the desired properties for universal cementation: high radiopacity, high mechanical strength, high bond strength, low film thickness and easy clean-up.

ALL-BOND UNIVERSAL™

ALL-BOND UNIVERSAL™ is the first TRULY UNIVERSAL dental adhesive that combines etching, priming and bonding in ONE bottle.

Z-PRIME™ plus

Z-PRIME™ Plus is a single-component priming agent used to enhance adhesion between indirect restorative materials and resin cements. It is a versatile primer for surface treatment of zirconia and alumina ceramics as well as metals/alloys, composites and endodontic posts.

PORCELAIN PRIMER

PORCELAIN PRIMER is a silane coupling agent used to improve the chemical bond between porcelain restorations and resin cements resulting in increased bond durability over time.

* It is recommended to use BISCO's CHOICE™ 2 for veneer cementation.